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ENGLISH - Instructions for the use of soft contact lenses

IMPORTANT WARNINGS

These instructions were written to provide you with some useful information in addition to your specialist's advice. The Evolens contact lens can be for one month, two months, three months, six months and one year duration. To know their use to see the indication on the label.

INSTRUCTIONS FOR THE USE AND STORAGE OF SOFT CONTACT LENSES: The information given in this leaflet is indispensable to be able to use the soft contact lenses safely and effectively. Read it carefully and use it as a guide together with the indications given by your specialist. The following instructions apply to all soft contact lenses.

GENERAL INSTRUCTIONS: these lenses were given to you exclusively for use during the daytime (take them out before you go to bed). Use sterile contact lenses and do not wear them for longer than the recommended period. At the end of the period of use recommended on the package (one day, two weeks, one month, etc.), the lenses must be replaced with a new pair. In order to use the contact lenses safely, the instructions for use are to be followed carefully in order to insert, remove, clean and maintain them properly.

The unopened lens package guarantees that the product is sterile; if the package is not sealed, **DO NOT USE** the lenses and return the package to the specialist who gave it to you.

The label on the container bears following information. BC/Rb= Curvature of rear surface, DIA/= Diameter of lens, PWR/D Corrective power in dioptres. For toric lenses AX= cylinder axis, CYL= cylinder, for multifocal lenses Add, \square = expiry date, month and year; \square = production lot for traceability, disposable lenses, do not use more than once, \square = sterilized in autoclave, \triangle **WARNING**= see the instructions. The trademark and CE number indicate that the product conforms to the standards laid down in the Medical Devices Directive.

Before inserting the contact lenses: Wash and dry your hands before removing the lenses from their container.

Insertion: Remove the lens from its container, making sure that it is not inside out and put it on the tip of your index finger. Before inserting the lenses, wash them with saline or multi-use solution. Standing in front of a mirror, pull down the lower eyelid with the middle finger of the same hand, (pic 3). With the middle finger of your free hand, pull up the upper eyelid and place the lens on your iris. (pic 4). Release the eyelid slowly. Repeat the operation for the other eye.

Vision: Your vision should be good as soon as you insert the lens, if it is not, check that the lens is not inside out and check that you have not inverted the lenses. (pic 1 and 2).

Comfort: If the lens feels uncomfortable, remove it, rinse it and insert it again. If you have inserted the lens inside out, remove it, flip it the right way round and clean it before putting it back in. **IF THE LENS CONTINUES TO FEEL UNCOMFORTABLE, REMOVE IT AND DO NOT REINSERT IT BEFORE CONSULTING YOUR SPECIALIST.**

Removal: Pull down the lower eyelid, look upwards and, using the tip of your finger, slide the lens to the "white" part of the eye. The lens may then be removed using your thumb and index finger. **TAKE CARE** not to scratch the surface of the lenses with your fingernails. Clean and store the lenses following the indications given to you by your specialist.

Remember to clean and replace the lens holder regularly (every 30 days) to prevent contamination.

Contraindications: The contact lenses may only be inserted and used when your eyes are in good anatomofunctional conditions. In fact, there are some risk factors, which, if present, will be identified by your specialist and may lead to complications and the onset of intolerance reactions. The specialist and the optician who inserts the lens are aware of these problems and will not recommend the use of contact lenses until you have passed a rigorous eye examination. In order to avoid harm to the eyes, it is important to check with the eye specialist that there are no contraindications and have periodic check-ups.

IMPORTANT NOTES

- Remove the lenses and consult a doctor if you suffer redness, a burning feeling, a foreign body sensation, excessive lacrimation, blurred vision or any other visual disorder.
- Diuretics, antihistamines, decongestants and tranquillizers may cause dry eyes, in which case you should consult your eye specialist.
- If a chemical substance comes into contact with your eyes, rinse them immediately and consult your doctor. Always make sure that the contact lens solution you are using has not expired whenever you take your lenses out and never rinse them under running water.
- Never put the lenses in your mouth to dampen them.
- Consult a doctor for advice on how to use them when playing sport.
- Avoid wearing contact lenses when you bathe in the sea or a swimming pool or use public showers.
- Avoid using soap that contains creams, lotions or cosmetic oils before inserting the lenses.
- Failure to follow the instructions for use of the contact lenses may cause severe injury to your eyes.
- Corneal ulcers that lead to visual impairment may occur in rare cases.
- The risk of contracting ulcerative keratitis increases if you fail to respect the rules of hygiene and use and if the lenses are used for a longer period than recommended.
- The risk of ulcerative keratitis is considerably higher in smokers.
- Contact lenses with UV protection are not a replacement for sunglasses because they do not cover the entire anterior segment of the eye so contact lens wearers must continue to wear sunglasses when exposed to UV light.

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